


HOW TO USE | ProTector™ Needle Sheath Prop



1. Assemble syringe with anesthetic and capped needle as usual.



2. Holding assembled syringe, insert cap through hole in Needle Sheath Prop, aligning ridges of needle cap with grooves around hole.

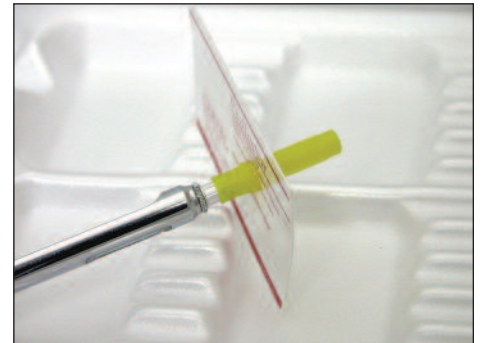

3. Insert needle cap through hole up to collar of cap.




4. Hold onto cap behind Protector Prop and pull cap off.



5. Place ProTector Prop on tray with open end of cap facing up.



6. Use one-handed technique to recap. Insert needle into cap; bring syringe vertical, push down slightly.

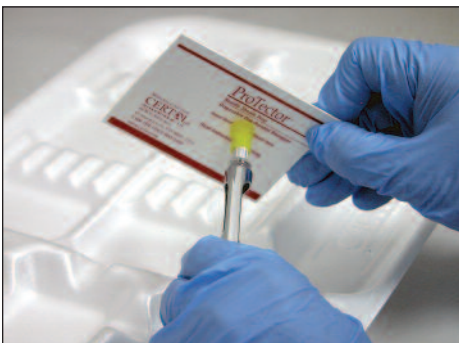

7. Disassembly: Leave ProTector Prop in place on cap. Hold edge of ProTector Prop to unscrew cap and needle as a unit. Keep ProTector Prop on cap/needle unit.



8. Place ProTector Prop parallel to surface of sharps container opening with exposed short end of needle pointing down into container.



9. Push cap out of ProTector Prop hole and let fall into sharps container.