

ProWash™

Multi-Purpose Instrument Detergent and Cart Wash

Low-Foaming / Neutral pH



Robust formula for wide variety of applications.

ProWash is a premium neutral pH multi-purpose detergent combining a powerful cleaning system with excellent material compatibility.

Effective Cleaning Action:

- Objective lab testing verifies effective cleaning action with challenging clinical conditions.
- Formulated for use in a broad range of water conditions with chelating and rinsing agents to ensure best results.
- Low-foaming formula is tested for effective use in automated instrument washers.

Protects Valuable Instrumentation:

- Broad material compatibility includes anodized aluminum, stainless steel, carbon steel and other common materials used in medical instrumentation.
- Non-corrosive formula.



ProWash Description	Case Qty	Order#
1 Gallon	4	PW128
5 Gallon Drum	1	PW050
15 Gallon Drum	1	PW150
30 Gallon Drum	1	PW300
55 Gallon Drum	1	PW550

ProWash™

Multi-purpose Instrument Detergent and Cart Wash

Description:

ProWash is a neutral pH, low-foaming, multi-purpose concentrated instrument detergent. Testing confirms a broad range of material compatibility including anodized aluminum, stainless steel and glass or lensed items. ProWash is a non-enzymatic detergent suitable for manual cleaning, ultrasonic instrument cleaning devices, instrument washers and cart washers.

Product Specifications:

Dilution Ratios:	1/2 oz. per gallon water for manual and ultrasonic cleaning. 1/2 oz. per gallon water for automated washers and cart washers depending on soils and water quality.
pH:	7.0 to 9.0
Active Ingredients:	Surfactants and cleaning agents, metal surface conditioning agents, chelating, anti-redeposition and anti-corrosive agents.
Temperature of Use:	In manual applications use 90° to 120°F (32° to 49°C). In automated systems follow washer manufacturer's instructions or settings. Best results obtained at 120° to 150°F (49° to 65°C).
Solution Color:	Blue.
Odor:	Non-specific.
Shelf Life:	3 year shelf-life, expiration date printed on product.

Product Capabilities:

Surfactants and other agents in ProWash act as a chemical team to aid in removing soils during instrument decontamination while protecting surface quality of instrumentation.

ProWash is low-foaming and ideal for use in automated washer and cart washer systems. It is also suitable for use when a non-enzymatic detergent is preferred.

ProWash is useful for cleaning ophthalmic instruments and devices when manufacturer's instructions recommend a neutral detergent formulated without enzymes.

Use Precautions:

ProWash detergent is a mild irritant. Wear appropriate protective equipment during decontamination procedures to protect against both chemical and biological hazards including eye protection and water-resistant gloves.

Accessory Products:

Other products available from Certol include:



ProEZ foam™
Ready-to-Use Foaming Enzymatic Spray



ProEZ AW Quad™
Quadruple Enzyme Automatic Washer Detergent



ProLube™ Concentrate
Instrument Lubricant Concentrate



ProLube™ RTU
Instrument Lubricant Ready-to-Use