

# ProChek® ID

## Steam Sterilization Indicator Tape



### Monitor Every Package

ProChek ID Process Indicator Tape is designed to be placed on every package that goes through the sterilizer to demonstrate that processing has occurred.

### Specifically Manufactured For Use In Steam Sterilizers.

- Tape adheres to any kind of material.
- Use on metal, glass, cloth, paper, nylon film, poly film, etc.

### Easy to Interpret

- Indicators turn from white to black.
- Distinguish processed from unprocessed packs.



ProChek ID Description	Case Qty	Order #
1/2" x 60 Yards	6	PIT120
3/4" x 60 Yards	6	PIT340
1" x 60 Yards	6	PIT100

## ProChek® ID

### Steam Sterilization Indicator Tape

#### Description:

ProChek ID Steam Sterilization Indicator Tape is an adhesive tape printed with a white indicator ink that turns dark when processed.

#### Intended Use:

ProChek ID Steam Sterilization Indicator Tape is used to seal sterilization packages and differentiate processed from unprocessed packages.

#### Product Specifications:

Paper Material: Specially treated crepe paper.  
Adhesive: Heat and water resistant adhesive.  
Indicator Ink: Steam specific indicator ink.  
Packaging Sizes: 1 roll/zip lock poly bag.

#### Product Capabilities:

ProChek ID Tape adheres to any kind of material (metal, glass, cloth, paper, nylon film, poly film) by simply pressing. Upon proper exposure to sterilization conditions, black diagonal lines will appear to show that the package has been processed. The white indicator stripes turn dark when exposed to 121°C (250°F) for 10 minutes, or 135°C (275°F) for 3 minutes.

---

#### Accessory Products:

Other products available from Certol include:




**ProView® plus**  
Self Sealing  
Sterilization Pouches



**ProPak™**  
Nylon Sterilization Tubing



**ProChek® S**  
Steam Sterilization  
Integrator

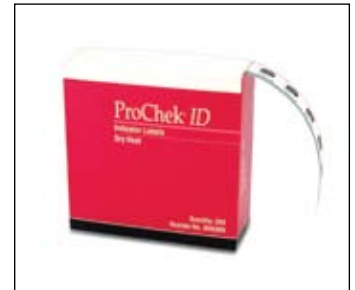

**ProChek® ID**  
Indicator Labels for Dry Heat