

Lemon
Scent

ProSpray™

Ready-to-Use Surface Disinfectant/Cleaner



ProSpray is a ready-to-use surface disinfectant and cleaner in one product, approved for use in medical and dental settings.

Broad Spectrum Disinfecting Power:

- Inactivates benchmark *M. tuberculosis* var. *bovis* and many other organisms.
- Meets CDC guidelines for surface disinfection in the presence of blood and other soils.

Also Effective Against:

- **Viruses including:** HIV-1, Hepatitis B and C, and Herpes simplex virus type 1 and 2 (see Note 1).
- **Vegetative bacteria including:** *Staphylococcus aureus*, *Salmonella typhosa*, and *Pseudomonas aeruginosa*.
- **Fungi including:** *Trichophyton mentagrophytes*.

Four Applications in One Product:

- *Surface cleaner:* Contains detergents and surfactants for essential pre-cleaning.
- *Intermediate level disinfectant.*
- *Ultrasonic instrument cleaning and holding solution.*
- *Disinfect dental impressions and appliances.*

Water Based Formula with Non-Alcohol Actives

- Water based formula preferred for a wide variety of surfaces including vinyl, metals, plastics, tubing and faux leather upholstery.
- Will not stain or bleach hard surfaces.
- Stays wet on surfaces longer for full contact time.
- Use around electronics is consistent with manufacturers' recommendations for non-alcohol, non-bleach formula.

Hard on Germs – Not on Bare Hands!

- So safe it is now EPA approved for use without gloves (in absence of biohazards).

Convenient and Pleasant to Use:

- Light lemon scent.
- Reduce inventory - one product cleans and disinfects.
- Ready-to-use: No mixing required.

Residual Activity

- Sustained antimicrobial action after application which is not found with alcohol or bleach compounds.

Economical Choice:

- One gallon and five gallon refills available.



ProSpray Description	Case Qty	Order #
24 oz. Spray Bottle	15	PSC240
1 Gallon Refill	4	PSC128
5 Gallon Refill	1	PSC050
Intro Kit - (2) 1 gallon refills, (4) 16 oz. empty spray bottles, 1 funnel, (4) squirt caps	1	PSCSYS
16 oz. Empty Spray Bottle	6	PSCPS

ProSpray™

Ready-to-Use Surface Disinfectant/Cleaner

Description:

ProSpray is a ready-to-use surface disinfectant and cleaner in one product, approved for use in medical and dental settings.

Intended Use:

ProSpray is a broad spectrum antimicrobial disinfectant and cleaner. It is tuberculocidal and meets CDC guidelines for intermediate level disinfectant use when blood and body soils are likely to be present. Detergents in the water based formula provide effective cleaning. It may be used on surfaces, as a cleaning solution for instruments and as a disinfectant for dental impressions and appliances.

Product Specifications:

Active Agents: 0.28% o-phenylphenol
0.03% o-benzyl-p-chlorophenol
EPA Reg. #: 46851-5
DIN #: 02231483
Solution Color: Natural color.
Odor: Lemon scent.
Shelf Life: 2 years from date of manufacture.

Product Capabilities:

Kills a broad range of microorganisms including HIV-1 in 60 seconds and the following in ten minutes: Escherichia coli 0113, staphylococcus aureus, Salmonella typhosa, Pseudomonas aeruginosa, M. tuberculosis var. bovis, fungi including Trichophyton mentagrophytes, viruses including Hepatitis B and C, influenza virus type A1 (New Jersey) and influenza B (Maryland), Herpes virus types 1 and 2, and Vaccinia virus.

Note 1: "Potency against M. tuberculosis has been recognized as a substantial benchmark. Any germicide with a tuberculocidal claim on the label is considered capable of inactivating a broad spectrum of pathogens, including such less resistant organisms as bloodborne pathogens (e.g., Hepatitis B virus, Hepatitis C virus, and HIV)."

CDC Guidelines for Infection Control in Dental Healthcare Settings – 2003. MMWR 2003;52(No.RR-17): p. 63.

Cleans and Disinfects:

Because effective disinfection relies on effective removal of soil, ProSpray is ideal for both steps – no need for two separate products. Special detergents in ProSpray clean even dried on blood and other soils.

Safe for Surfaces:

ProSpray is a water based disinfectant with non-alcohol actives. Water based disinfectant formulas are preferred by upholstery manufacturers. And, unlike alcohol based formulas, ProSpray will not soften or haze vinyls and plastic.

ProSpray will not corrode metals or stainless steel counters as may occur with bleach based products. It will not stain or bleach surfaces even with repeated use.

How to Use:

For surface disinfection – use the 2 step method.

- Clean by wiping surfaces thoroughly with ProSpray and paper towels or 4 x 4 gauze to remove all soils.
- Disinfect by reapplying ProSpray disinfectant, keeping surfaces wet for 10 minutes. Wipe off excess solution after contact time is completed.

For instrument holding and cleaning solution:

- Fill pan or ultrasonic tank with ProSpray solution. Immerse items so that all surfaces are covered with solution. Soak for a minimum of 10 minutes. Do not soak items more than two hours or overnight due to corrosive nature of tap water.

For disinfecting dental impressions and appliances:

- Rinse and remove gross debris from item. Place impression or appliance inside baggie, thoroughly wet all surfaces with ProSpray, allow 10 minute contact time, remove item and rinse gently.

Use Precautions:

A slight soap film may occur upon repeated application of ProSpray due to detergents in the formula. Simply wipe surfaces off at least weekly with plain water and paper towels.

Rinse garments or surfaces where ProSpray is used prior to application of any bleach product.

Store solution away from heat and direct sunlight. Avoid freezing during shipping or storage.

Accessory Products:

Other products available from Certol include:



ProSpray C-60™
Concentrated Surface
Disinfectant/Cleaner



ProSpray™ wiper
Disinfectant Towelettes



ProTouch® vinyl
Vinyl Examination Gloves



Good Vibrations™
Ultrasonic Cleaner
Solution