

ProSpray™ wipes

Surface Disinfectant/Cleaner Towelettes

Ready-to-Use



ProSpray wipes are a ready-to-use surface disinfectant and cleaner in one product, approved for use in medical and dental settings.

Broad Spectrum Disinfecting Power:

- Inactivates benchmark *M. tuberculosis* var. *bovis* and many other organisms.
- Meets CDC guidelines for surface disinfection in the presence of blood and other soils.

Also Effective Against:

- Viruses including: HIV-1, Hepatitis B and C, and Herpes simplex virus type 1 and 2 (see Note 1).
- Vegetative bacteria including: *Staphylococcus aureus*, *Salmonella typhosa*, and *Pseudomonas aeruginosa*.
- Fungi including: *Trichophyton mentagrophytes*.

Applications:

- Surface cleaner: Contains detergents and surfactants for essential pre-cleaning.
- Intermediate level disinfectant.

Water Based Formula with Non-Alcohol Actives:

- Water based formula preferred for a wide variety of surfaces including vinyl, metals, plastics, tubing and faux leather upholstery.
- Will not stain or bleach hard surfaces.
- Stays wet on surfaces longer for full contact time.

Hard on Germs – Not on Bare Hands!

- So safe it is now EPA approved for use without gloves (in absence of bio-hazards).

Convenient and Pleasant to Use:

- Light lemon scent.
- Reduce inventory: One product to clean and disinfect.
- Pre-saturated with correct amount of solution.
- Non-abrasive material distributes disinfectant evenly.
- Eliminates aerosols created by spray disinfectants.

Residual Activity:

- Sustained antimicrobial action after application which is not found with alcohol or bleach compounds.

Ideal for Disinfection of Electronic Devices:

- Use around electronics is consistent with manufacturers' recommendations for non-alcohol, non-bleach formula.
- Use on telephones, keyboards, cords and digital radiographic sensors.*



ProSpray wipes Description	Case Qty	Order#
Soft Pack - 9" x 10" - 72/soft pack	12	PSW
Canister - 6" x 6.75" - 240/canister	12	PSWC
Tall Canister - 8.5" x 12" - 135/canister	12	PSWCT
Canister Bracket (fits both regular and large canisters)	1	PSWCR

* Consult manufacturer regarding application of ProSpray wipes on specific electronic devices.

ProSpray™ wipes Surface Disinfectant/Cleaner Towelettes

Description:

ProSpray wipes are a ready-to-use surface disinfectant and cleaner in one product, approved for use in medical and dental settings.

Intended Use:

ProSpray wipes are a broad spectrum antimicrobial disinfectant and cleaner. They are tuberculocidal and meet CDC guidelines for intermediate level disinfectant use when blood and body soils are likely to be present. Detergents in the water based formula provide effective cleaning.

Product Specifications:

Active Agents: 0.28% o-phenylphenol
0.03% o-benzyl-p-chlorophenol
EPA Reg. #: 46851-10
Solution Color: Natural color
Odor: Lemon scent
Shelf Life: 2 years from date of manufacture

Product Capabilities:

Kills a broad range of microorganisms including HIV-1 in 60 seconds and the following in ten minutes: Escherichia coli 0113, Staphylococcus aureus, Salmonella typhosa, Pseudomonas aeruginosa, M. tuberculosis var. bovis, fungi including Trichophyton mentagrophytes, viruses including Hepatitis B and C, influenza virus type A1 (New Jersey) and influenza B (Maryland), Herpes virus types 1 and 2, and Vaccinia virus.

Note 1: "Potency against M. tuberculosis has been recognized as a substantial benchmark. Any germicide with a tuberculocidal claim on the label is considered capable of inactivating a broad spectrum of pathogens, including such less resistant organisms as bloodborne pathogens (e.g., Hepatitis B virus, Hepatitis C virus, and HIV)." CDC Guidelines for Infection Control in Dental Healthcare Settings – 2003. MMWR 2003;52(No.RR-17): p. 63.

Cleans and Disinfects:

Because effective disinfection relies on effective removal of soil, ProSpray wipes are ideal for both steps – no need for two separate products. Special detergents in ProSpray wipes clean even dried on blood and other soils.

Safe for Surfaces:

ProSpray wipes are a water based disinfectant with non-alcohol actives. Water based disinfectant formulas are preferred by upholstery manufacturers. And, unlike alcohol based formulas, ProSpray wipes will not soften or haze vinyls and plastic.

ProSpray wipes will not corrode metals or stainless steel counters as may occur with bleach and hydrogen peroxide based products. They will not stain or bleach surfaces even with repeated use.

How to Use:

For surface disinfection – use the 2 step method.

- To clean, remove one or more wipe towelettes and wipe surfaces thoroughly to remove all soils. Discard used wipe towelettes.
- Remove one or more additional fresh wipe towelettes. Reapply disinfectant to previously cleaned surfaces for a 10 minute contact time.

Use Precautions:

A slight soap film may occur upon repeated application of ProSpray wipes due to detergents in the formula. Simply wipe surfaces off at least weekly with plain water and paper towels.

Rinse garments or surfaces where ProSpray wipes are used prior to application of any bleach product.

Store ProSpray wipes away from direct heat and direct sunlight. Avoid freezing during shipping or storage.

Accessory Products:

Other products available from Certol include:



ProSpray C-60™
Concentrated Surface
Disinfectant/Cleaner



ProSpray™
Ready-to-Use Surface
Disinfectant/Cleaner



ProSpray™ wipes
Canister Bracket
Fits All Certol Canisters



Good Vibrations™
Ultrasonic Cleaner
Solution