

# ProClense™ plus

## Instrument Detergent

### Low-Foaming



*Economical and effective cleaning solution.*

ProClense plus is an economical detergent with effective cleaning agents to help maintain instrument surface quality.

#### Effective Cleaning Action:

- Objective lab testing verifies effective cleaning action with challenging clinical conditions.
- Formulated for use in a broad range of water conditions with chelating and rinsing agents to ensure best results.
- Low-foaming formula is tested for effective use in automated instrument washers.

#### Protects Valuable Instrumentation:

- Broad material compatibility includes anodized aluminum, stainless steel, carbon steel and other common materials used in medical instrumentation.
- Non-corrosive formula.
- No added dyes.



ProClense plus Description	Case Qty	Order#
1 Gallon	4	PCNF128
5 Gallon Drum	1	PCNF050
15 Gallon Drum	1	PCNF150
30 Gallon Drum	1	PCNF300
55 Gallon Drum	1	PCNF550

## ProClense™ plus Instrument Detergent

### Description:

ProClense plus is an economical, low-foaming, multi-purpose concentrated instrument detergent. Testing confirms a broad range of material compatibility including anodized aluminum, stainless steel and glass or lensed items. ProClense plus is a non-enzymatic detergent suitable for manual cleaning, ultrasonic instrument cleaning devices, instrument washers and cart washers.

### Product Specifications:

Dilution Ratios:	1/2 oz. per gallon (4 ml per liter) of warm water for manual, ultrasonic and automated washers and cart washers.
pH:	8.0 to 10.0
Active Ingredients:	Surfactants and cleaning agents, metal surface conditioning agents, chelating, anti-redeposition and anti-corrosive agents.
Temperature of Use:	In manual applications do not exceed 120°F (49°C). In automated systems follow washer manufacturer's instructions or settings. Best results obtained at 120° to 150°F (49° to 65°C).
Solution Color:	Clear.
Odor:	Non-specific.
Shelf Life:	3 year shelf-life, expiration date printed on product.

### Product Capabilities:

Surfactants and other agents in ProClense plus act as a chemical team to aid in removing soils during instrument decontamination while protecting surface quality of instrumentation.

ProClense plus is low-foaming and ideal for use in automated washer and cart washer systems. It is also suitable for use when a non-enzymatic detergent is preferred.

### Use Precautions:

ProClense plus detergent is a mild irritant. Wear appropriate protective equipment during decontamination procedures to protect against both chemical and biological hazards including eye protection and water-resistant gloves.

### Accessory Products:

Other products available from Certol include:



**ProEZ foam™**  
Ready-to-Use Foaming  
Enzymatic Spray



**ProEZ AW Quad™**  
Quadruple Enzyme  
Automatic Washer  
Detergent



**ProLube™ Concentrate**  
Instrument Lubricant  
Concentrate



**ProLube™ RTU**  
Instrument Lubricant  
Ready-to-Use