

# ProEZ AW Quad™

## Concentrated Quadruple Enzyme Automatic Washer Detergent



### **Powerful, Non-Foaming, Four Enzyme Cleaning Formula.**

ProEZ AW Quad is a premium, neutral pH, four enzyme instrument cleaning detergent specifically designed for use in high impingement systems. Enzymes and surfactants work as a team to handle tough cleaning challenges.

- Highly concentrated.
- Biodegradable.
- Anti-corrosive agents.
- Instrument brightener.
- Safe for all surgical instruments and scopes.
- Indicated for use in the cleaning cycle of automated scope reprocessors.

### **Four Powerful Cleaning Enzymes:**

- Protease, amylase, lipase and cellulase enzymes work together to quickly dissolve protein, starch and fatty soils in as little as one minute.
- Ideal formulation for your toughest cleaning challenges such as orthopedic and gastrointestinal soils.
- Thorough cleaning for complex instrument surfaces including cannulated areas, box locks and serrations.

### **Superior Cleaning System:**

- Carefully selected surfactants reduce surface tension, improve penetration of cleaning agents and promote free rinsing of dissolved soils.

### **Multi-Use Detergent:**

- Simplify purchasing and inventory. Standardize on one highly effective formula.
- Ideal for high impingement automated washer/disinfectors and washer/sterilizers.
- No foam formula also indicated for ultrasonic cleaning systems including automated washers for cannulated items.
- Fast, effective cleaning of tough gastrointestinal soils in automated endoscope washers.
- No foam formula improves visibility for increased safety during manual/pan soaking and flushing.

### **Cost Effective:**

- ProEZ AW Quad is highly concentrated with effective dilution starting as low as 1/2 ounce per gallon. Adjust dilution based on soil load and water quality.



<b>ProEZ AW Quad Description</b>	<b>Case Qty</b>	<b>Order#</b>
1 Gallon	4	PREZAW128
1 Gallon Re-Order w/o Pump	4	PREZAWN128
5 Gallon Drum	1	PREZAW050
15 Gallon Drum	1	PREZAW150
30 Gallon Drum	1	PREZAW300
55 Gallon Drum	1	PREZAW550

## ProEZ AW Quad™

### Concentrated Quadruple Enzyme Automatic Washer Detergent

#### Description:

ProEZ AW Quad is a four enzyme, non-foaming, neutral pH detergent containing powerful surfactants for use in automatic washers. Protease, amylase, lipase and cellulase enzymes work together with carefully selected surfactants to clean surgical instruments and endoscopes.

#### Product Specifications:

Dilution Ratios: 1/2 oz. to 1 oz. per gallon.  
pH: Neutral.  
Temperature of Use: ProEZ AW Quad has an expanded temperature range. It is effective at room temperature about 68°F (20°C). Use with warmer water at 90-110°F (32-43°C) will optimize the performance of the enzymes. This product may be used in automated cleaning equipment (ultrasonics, AEDs, instrument washers) from 90-150°F (32-65°C) according to equipment manufacturer's instructions.

Odor: Fresh.  
Color: Blue.  
Shelf Life: 2 years from date of manufacture.

#### Intended Use:

ProEZ AW Quad is specially formulated for use in high impingement automated washers and washer/sterilizers. ProEZ AW Quad is intended for cleaning medical instruments by dissolving and lifting organic soils in as little as 1 minute. It is specifically designed for cleaning instruments with difficult to access areas such as endoscope channels, box locks and serrated edges.

#### Product Capabilities:

ProEZ AW Quad rapidly digests protein, starch and fats, the principle components of surgical soils, and is non-corrosive and safe to use on all instruments.

It is a convenient 4-in-1 product which may be used:

1. As an instrument holding solution.
2. To clean instruments in manual systems.
3. In ultrasonic cleaners and scope reprocessors.
4. In automatic washers.

The non-foaming solution aids in the visualization of instruments when used in manual systems.

#### Use Precautions:

As with other enzymatic detergents, ProEZ AW Quad is an eye irritant and mild skin irritant. Wear eye protection and water resistant utility gloves when using the product.

#### Accessory Products:

Other products available from Certol include:



**ProEZ foam™**  
Ready-to-Use Foaming  
Enzymatic Spray



**ProWash™**  
Multi-Purpose  
Instrument Detergent  
and Cart Wash



**ProClense™ plus**  
Instrument Detergent



**ProLube Concentrate™**  
Instrument Lubricant  
Concentrate



**ProLube RTU™**  
Instrument Lubricant  
Ready-to-Use