

SUPER CONCENTRATED DETERGENT LINE

Certol[®]
International

ProWash SCD™

Super Concentrated Instrument Detergent and Cart Wash

Non-Foaming • Neutral pH Formula • Anti-Corrosive



*Premium neutral pH
multi-purpose formula*

ProWash SCD is part of the:

SUPER CONCENTRATED DETERGENT LINE

SUPER SYSTEM



Enzymatic



Neutral



Alkaline



Lubricant

Super Concentrated:

- Clinically tested for effective cleaning at ultra low dose settings.
- Reduces weight of containers and storage space.
- Delivers critical cleaning performance with reduced cost per cycle.

Broad Material Compatibility:

- Neutral pH meets manufacturer's instructions for many instruments and devices.
- Extensive testing demonstrates compatibility with aluminum trays and containers, gold, brass, copper, stainless steel and titanium.
- Active anti-corrosive system.

Enhanced Cleaning Action in Automated Equipment:

- Chemical engineering to enhance cleaning action at neutral pH.
- Non-foaming for easy rinsing.
- Promotes ideal pump pressure and spin action for best cleaning action.
- Chelating agents ensure best performance over wide range of operating conditions.



ProWash SCD Description	Case Qty	Order#
2.5 Gallon Drum	1	PWSCD2.5

ProWash SCD™

Super Concentrated Instrument Detergent and Cart Wash

Description:

ProWash SCD is a neutral pH detergent used when cleaning instruments and devices in automated equipment. It is super concentrated to reduce package weight and storage space.

Product Capabilities:

ProWash SCD contains surfactants and other components to rapidly dissolve a wide range of surgical soils. The neutral pH formula complies with manufacturer's instructions when specifying neutral pH and non-corrosive formulas. It is appropriate for use on heat tolerant instruments and devices indicated for cleaning in automated washing equipment.

Product Specifications:

Dilution Ratios: 1/40 to 1/5 oz. per gallon
(0.2 ml to 1.6 ml per liter).
pH: Neutral.
Temperature of Use: 120°F to 150°F (49°C to 66°C).
Odor: Non-specific, fresh.
Color: Natural clear yellow.
Shelf Life: 2 years from date of manufacture.

Intended Use:

ProWash SCD Super Concentrated Detergent and Cart Wash is non-foaming and formulated to promote ideal pressure and spin action in automated washer/disinfectors and cart washers. This product is used to clean instruments and appropriate devices used in healthcare and other settings. The formula is neutral pH with advanced anti-corrosive components for broad material compatibility with stainless steel, titanium, brass, gold, copper, aluminum, anodized aluminum, heat tolerant plastics and rubber goods. All applications should be according to manufacturer's instructions.

Accessory Products:

Other products available from Certol include:



Certol Family of Super Concentrated Detergents

ProEZ SCD™

- QUAD Enzymatic

ProWash SCD™

- Neutral

ProClense plus SCD™

- Alkaline

ProLube SCD™

- Lubricant



ProSpray™ wipes

Surface Disinfectant/
Cleaner Towelettes



ProEZ foam™

Ready-to-Use Foaming
Enzymatic Spray