

V-Cide®

Chemical Vapor Sterilant Solution



V-Cide is a chemical sterilant formulated for use in Harvey® Chemiclave® sterilizers.



Reduced Odor:

- Improved formula reduces odor-causing agents.

Gentle and Non-Corrosive:

- Compatible for use in Chemiclave® sterilizers.
- Will not corrode, rust or dull instruments when used in Chemiclave® sterilizers according to manufacturer's instructions.

Economical:

- Cost-effective for your Chemiclave®.

Effective:

- Tested with FDA clearance to market as a sterilant solution for Chemiclave® sterilizers.

Compatible:

- Safe for use with materials recommended by manufacturers for sterilization in chemical vapor sterilizers.



V-Cide Description	Case Qty	Order#
Liter Bottle	16	VC338

V-Cide is a registered trademark of Certol International, LLC. V-Cide is not made by or affiliated with Harvey Chemiclave. Harvey and Chemiclave are registered trademarks of Thermo Scientific.

V-Cide®

Chemical Vapor Sterilant Solution

Description:

V-Cide is a liquid chemical sterilant solution for use in chemical vapor sterilizers.

Intended Use:

V-Cide is a sterilant solution designed for use in chemical vapor sterilizers including the Harvey® Chemiclave®, Chemical vapor sterilizers are designed and validated to ensure sterilization using only this special solution.

Product Specifications:

Active Ingredient: Formaldehyde, 0.23%;
Ethanol, 57.02%.
Solution Color: Clear.
Odor: Alcohol odor.
Shelf Life: 2 years.

Product Capabilities:

V-Cide significantly reduces the risk of corrosion and dull edges on instruments. V-Cide is safe for use with materials recommended by manufacturers for sterilization in chemical vapor sterilizers.

Use Precautions:

Dry instruments thoroughly before packaging and sterilization in chemical vapor sterilizer. (Residual water will interfere with sterilization and may contribute to corrosion.)

Use personal protective equipment (PPE) as directed on the product Safety Data Sheet: wear eye protection, chemical resistant gloves, mask and appropriate clothing to prevent repeated or prolonged skin contact. Avoid spilling V-Cide on exposed skin or splashing in eyes. V-Cide is flammable and should be stored away from sterilizers in a dry, cool area. Keep away from open flames. Use in a well-ventilated area. When running and opening the chemical vapor sterilizer at end of cycle, avoid direct inhalation of vapors. Keep container closed between uses.

Accessory Products:

Other products available from Certol include:



ProSpray™ wipes
Surface Disinfectant/
Cleaner Towelettes



ProView® plus
Self Sealing Sterilization
Pouch



Good Vibrations™
Ultrasonic Cleaner
Solution



ProE-Vac™
Liquid Evacuation System
Cleaner