

SUPER CONCENTRATED DETERGENT LINE

Certol[®]
International

ProWash SCD™

Super Concentrated Instrument Detergent and Cart Wash

Non-Foaming • Neutral pH Formula • Anti-Corrosive



**Premium neutral pH
multi-purpose formula**

Value and performance in super concentrated formula:

- 2.5 gallons of ProWash SCD delivers over 300 loads of clean instruments in a standard instrument washer or about 50 loads in cart washer.*
- 2.5 gallon container replaces 15 to 30 gallon drum.
- Reduced weight and footprint for easier handling and storage.

Standardize with a versatile detergent:

- Effective, economical cleaning in instrument washers AND cart washers.

Convert with confidence!

- Testing validates effective cleaning at ultra low dose points.
- Developed using ASTM D8179 guidelines for medical detergents.
- Neutral pH with superior anti-corrosive system offers broad material compatibility and instrument protection including anodized aluminum containers.

*Dosing at 1/10 oz./gal, 1 mL/liter or 0.1%.

ProWash SCD is part of the:

SUPER CONCENTRATED DETERGENT LINE

SUPER SYSTEM



Enzymatic



Neutral



Alkaline



Lubricant



ProWash SCD Description	Case Qty	Order#
2.5 Gallon Drum	1	PWSCD2.5

ProWash SCD™

Super Concentrated Instrument Detergent and Cart Wash

Description:

ProWash SCD is a neutral pH detergent used when cleaning instruments and devices in automated equipment. It is super concentrated to reduce package weight and storage space.

Product Capabilities:

ProWash SCD contains surfactants and other components to rapidly dissolve a wide range of surgical soils. The neutral pH formula complies with manufacturer's instructions when specifying neutral pH and non-corrosive formulas. It is appropriate for use on heat tolerant instruments and devices indicated for cleaning in automated washing equipment. Engineered formula enhances spin and cleaning action in all models and designs of automated washers and cart washers.

Product Specifications:

Dilution Ratios: 1/40 to 1/5 oz. per gallon
(0.2 mL to 1.6 mL per liter).
pH: Neutral.
Temperature of Use: 120°F to 150°F (49°C to 66°C).
Odor: Non-specific, fresh.
Color: Natural clear yellow.
Shelf Life: 2 years from date of manufacture.
Packaging: 2.5 gal. drum, 1 each.

Intended Use:

ProWash SCD Super Concentrated Detergent and Cart Wash is non-foaming and formulated to promote ideal pressure and spin action in automated washer/disinfectors and cart washers. This product is used to clean instruments and appropriate devices used in healthcare and other settings. The formula is neutral pH with advanced anti-corrosive components for broad material compatibility with stainless steel, titanium, brass, gold, copper, aluminum, anodized aluminum, heat tolerant plastics and rubber goods. All applications should be according to manufacturer's instructions.

Precautions:

Use Certol SCD super concentrates in cleaning equipment designed to dose at more precise volumes from 1/5 oz. per gal. to 1/20 oz. per gal. (0.2% – 0.05%).

Accessory Products:

Other products available from Certol include:



Certol Family of Super Concentrated Detergents

ProEZ SCD™

• QUAD Enzymatic

ProWash SCD™

• Neutral

ProClense plus SCD™

• Alkaline

ProLube SCD™

• Lubricant



ProEZ AW Quad™

Quadruple Enzyme
Automatic Washer
Detergent



ProEZ foam™

Ready-to-Use Foaming
Enzymatic Spray